

## Detection of Phosphorus in Human Serum with Roche c502

<b>Test Name:</b>	In vitro test for the quantitative determination of phosphorus in human serum and plasma on Roche/Hitachi cobas c systems.
<b>Method Name:</b>	Molybdate UV. The concentration of phosphomolybdate formed is directly proportional to the inorganic phosphate concentration and is measured photometrically.
<b>Results:</b>	Technical Range: 0.31-20 mg/dL Reportable Range: 0.34-13.91 mg/dL
<b>Reference Ranges:</b>	Taken from UnityPoint-Peoria. 2.5 - 4.9 mg/dL
<b>Clinical Significance:</b>	<p>88 % of the phosphorus contained in the body is localized in bone in the form of calcium phosphate as the apatite <math>\text{Ca}^{2+}[\text{Ca}_3(\text{PO}_4)_2]^{3^{2-}}</math>. The remainder is involved in intermediary carbohydrate metabolism and in physiologically important substances such as phospholipids, nucleic acids and ATP. Phosphorus occurs in blood in the form of inorganic phosphate and in organically bound phosphoric acid. The small amount of extracellular organic phosphorus is found almost exclusively in the form of phospholipids.</p> <p>The ratio of phosphate to calcium in the blood is approximately 6:10. An increase in the level of phosphorus causes a decrease in the calcium level. The mechanism is influenced by interactions between parathormone and vitamin D. Hypoparathyroidism, vitamin D intoxication and renal failure with decreased glomerular phosphate filtration give rise to hyperphosphatemia. Hypophosphatemia occurs in rickets, hyperparathyroidism and Fanconi's syndrome.</p>
<b>Submission Criteria:</b>	<p>For specimen collection and preparation, only use suitable tubes or collection containers. Only the specimens listed below were tested and found acceptable. Serum Plasma: Li-heparin and <math>\text{K}_2</math>-EDTA plasma Do not use fluoride plasma</p> <p>The sample types listed were tested with a selection of sample collection tubes that were commercially available at the time of testing, therefore not all available tubes of all manufacturers were tested. Sample collection systems from various manufacturers may contain differing materials which could affect the test results in some cases. When processing samples in primary tubes (sample collection systems), follow the instructions of the tube manufacturer.</p> <p>Storage and Stability: 24 hours at 15-25 °C 4 days at 2-8 °C 1 year at (-15) - (-25) °C</p>

**Rejection Criteria:**

Rejection criteria include but are not limited to:

1. Specimens containing fibrin or clots.
2. Excessive platelet clumping
3. Leaking specimens
4. Substandard mixing or collection
5. Expired or improperly stored collection tubes.
6. Improperly filled tubes based on collection tube manufacturer's guidelines.
7. Contaminated specimens (IV fluid, foreign particles, etc.)
8. Specimens not analyzed within the appropriate time frame.
9. Samples not shipped at appropriate temperature.
10. Samples without 2 proper identifiers or samples having identifiers that do not match the electronic or paper lab requisition.

**Authorization:**

Diagnostic testing can only be performed with approval from an authorized provider/agency.

**Turn Around Time:**

1 day.

## Instructions for Serum Specimen Submission

### General Information

The detection of phosphorus in human serum and plasma is performed using a Roche cobas i58 analyzer. Serum specimens are preferred.

Specimens must be collected and stored at 20-25°C if to be analyzed within 24 hours, at 2-8°C if to be analyzed within 4 days and stored at (-15°C) – (-20°C) if to be analyzed within 365 days. Please be aware that storing specimens at  $\leq -70^{\circ}\text{C}$  ( $\leq -94^{\circ}\text{F}$ ) is not permissible.

**Specimens MUST be received at Reditus Laboratories within 4 days of collection at 2-8°C.**

### Collection Instructions for Serum Specimen

1. Do not use expired collection tubes. Store collection tubes as per manufacturers recommendations. Use standard venipuncture practices for collecting samples. Filled gold top serum tubes are preferred.
2. Ensure that the patient's name, date-of-birth, and time/date of collection are recorded on the specimen tube along with the name or initials of the individual collecting the sample.
3. Complete all the demographic information on a sample requisition form through the approved electronic submission process
4. Refrigerate the specimen between 2-8°C (36-46°F) and ship or courier the specimen(s) within 72 hours.
5. The specimen(s) *must* be received at the laboratory **no later than 72 hours from the time of collection**.
  - a. **Avoid shipping specimens over weekends or holidays** as they may not be received at the laboratory and cold-packs will not maintain the required 2-8°C (36-46°F) specimen temperature.
  - b. Ensure that specimens shipped by commercial carrier are shipped with **overnight delivery**. If shipping on a Friday for Saturday delivery, ***you must include Saturday Delivery*** during shipment, or the specimens will not be received until the following non-holiday business day. Failure to receive specimens within 24 hours of shipment will result in specimens being rejected from testing.
6. For any questions pertaining to sample collection, storage, or shipping, please contact the Reditus Laboratories using the below contact information.

### Instructions for Specimen Transport

7. **Messenger/Courier by ground transport.** Place specimen(s) into a biohazard labeled bag and seal securely. Place the test requisition(s) on the outside of the biohazard labeled bag. Place the sealed biohazard bag and test requisition(s) inside the shipping container. Place cold packs, which have been frozen for at least 24 hours, in the leak-proof outer container. The shipping container must be rigid, such as a Styrofoam cooler, and labeled with the UN 3373 Biological Substance Category B marking. Close securely.
8. **Commercial carrier by ground/air transport.** Place the specimen(s) inside a biohazard labeled bag and seal securely. Place the test requisition(s) on the outside of the biohazard labeled bag. Place the sealed bag and completed test requisitions(s) inside the outer shipping container. Place cold packs, which have been frozen for at least 24 hours, in the leak-proof outer container. Label the outer shipping container with Reditus Laboratories address listed below. Complete the return address section to include the name of the person shipping the package, business name and address, and a business phone number. The shipping container must include the UN3373 Biological Substance Category B marking.
9. *Ship specimens by overnight delivery* to the attention of Clinical Chemistry at Reditus Laboratories. This can be accomplished by use of local courier, shipping corporations or U.S. Postal Service.
  - a. **If specimens are shipped on a Friday for Saturday delivery, *you must include/indicate Saturday delivery*** during shipment, or the specimens will not be received until the following non-holiday business day. Failure to receive specimens within 24 hours of

shipment will result in specimens being rejected from testing.

10. The specimen(s) must be received at the laboratory **no later than** 72 hours *from the time of collection* and 24 hours from the time of shipment. Do not ship specimens over weekends or holidays as they will not be received, and cold-packs will not maintain the required 2-8°C (36-46°F) specimen temperature.

**NOTE:** Testing may be delayed, or specimens may be considered UNSATISFACTORY if the above instructions are not followed or the requisition form is not filled out completely. If there are any questions about specimen collection, handling, or shipping please contact the Reditus Laboratories to speak with laboratory personnel.

Ship specimens by a local courier or overnight by commercial carrier to the designated laboratories indicated below.

**Send to:** Reditus Laboratories  
200 Enterprise Drive  
Pekin, IL 61554

**Phone:** (469) 498-0222

**Website:** <https://www.redituslabs.com/>